Electromagnetic Muscle Stimulation Machine Non Invasive RF Slim Beauty Machine 5000W For Weight Loss

Basic Information

Place of Origin: Shandong, ChinaBrand Name: KM healthy&Beauty

Model Number: KMslim

• Price: \$1,900.00/sets 1-9 sets

• Packaging Details: onychomycosis laser / 980nm diode laser

vascular spider vein removal with Aluminum

box



Product Specification

Type: Slimming MachineFeature: Weight Loss

• Plugs Type: AU, Other, UK, EU, US, CN, JP, Za, It

Application: For Commercial

• Target Area: Body

• Handles: 4 Piece Handle

Product Name: Slim Beauty Machine

Shape Of Stimulatio: Pulse: Transformed Wave

• Pulsation: 300us

Output Intensity: 0~13 Tesla
Product Size: 57*51*135cm

Power: 5000wFrequency: 150HZ

Keywords: EMS Body Sculpting Shape Slimming

System Machine



More Images










Product Description

EMS Muscle Electromagnetic Non-Invasive RF Slim Beauty Machine 5000w For Weight Loss

Products Description

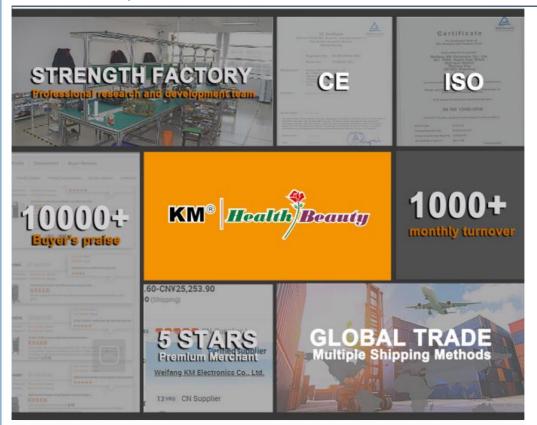



PRODUCT FUNCTION

SHOW YOU HOW OUR PRODUCTS WORK



- Non-invasive buttock lifting procedure.
- 3.Suitable for everyone-no anaesthesia
- -no surgery.
- 4.Only a 30minute walk in walk out procedure.
- 5.Only 4 session needed 2-3 days apart.
- 6.feels like an intensive workout.
- 7. Safe with no downtime.
- 8.Instant results but gets better after two to four weeks.







Arms





ABS

Buttocks

Thighs



Function

* BUILD MUSCLE

The muscle contracts 30000 times withhigh frequency and intensity, so asto train and increase muscledensity and volume

* REDUCE FAT

The ultimate contraction of muscle needsa large amount of energy supply, so thefat cells beside the muscle are alsoconsumed, leading to natural apoptosis and effective reduction of fatthickness

* Muscle Sculpt

Exercising the abdominal muscles, shaping the vest line / exercising the hip muscles, creating the peach hips / exercising the abdominal oblique muscles, and shaping the mermaid line.

Four handles, included special handles for arms





Theory

WORK THEORY of EMSlim machine


Non-invasive EMS procedure induces powerful muscle contractions notachievable through voluntary contractions. When exposed to strong contractions, the muscle tissue is forced to adapt to such extreme condition, It responds with a deep remodeling of its inner structure that results inmuscle building and sculpting your body.

At the same time, the 100% extreme muscle contraction of EMS technology can trigger a large amount of fat.decomposition, excreted by the body's normal metabolism within a few weeks, Therefore, slim beauty machine can strengthen and increase muscle, and reduce fat at the same time.

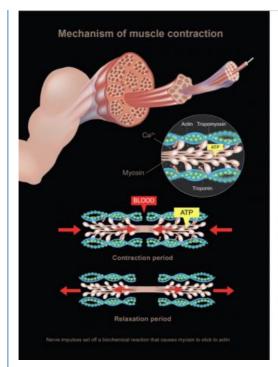

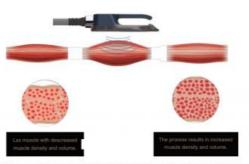


Ems. Pro induces very strong (supramaximal) muscle contractions,not achievable through workout. When exposed to these strong contractions,the muscle tissue is forced to adapt to such extreme condition. The musle starts remodelling its inner structure by multiplication of fibers and their growth. The HIFEM uses a specific range of frequencies that does not allow muscle relaxation between two consecutive stimulations. The muscle is forced to remain in contracted state for multiple seconds. When repeatedly exposed to these high load conditions the muscle tissue is stressed and is forced to adapt. Recent studies reported that on average 15%-16% increase inabdominal muscle thickness was observed in treated patients one to two months after HIFEM treatments.

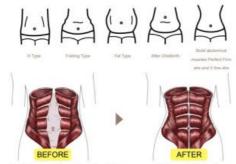

The Emsi Pro works on the abdomen area contracting and tightening the muscles to both burn fat and sculpt the area. This treatment can help you improve the muscle definition that you are already seeing from your fitness routine. It is also ideal for addressing that stub-



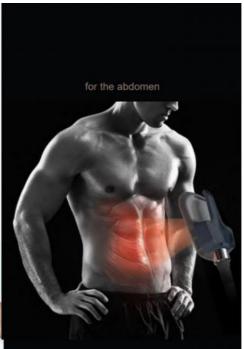





Patented cooling technology makes

Continuous cooling technology makes the treatment head will not produce high tem-perature, greatly improving its output power which make it very stable.



The market magnetic wave instruments are generally only 2.5-3.0 Tesla, while the magnetic thinness can be adjusted up to 7 Tesla, The perfectly balanced continuousenergy output rate is absolutely speechless in the body building equipment on the market.

Operating placement

- Body shaping
- Muscle Building
- Abdomen sculpting

- Buttocks lifting
- Thighs shaping














Easy Operate .


5 modes of movement

- *HIIT- High intensity interval training mode of aerobic fat reduction.
- *Hypertrophy --Muscle strengthening training mode
- *Strength -- Muscle strength training mode
- *HIIT+ Hypertrophy Training mode of strengthening muscle &reducing fat
- *Hypertrophy + + Strength Training mode of strengthening muscle&muscle strength

High Efficiency

30 minutes treatments= 20000 workouts

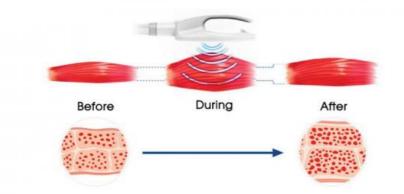

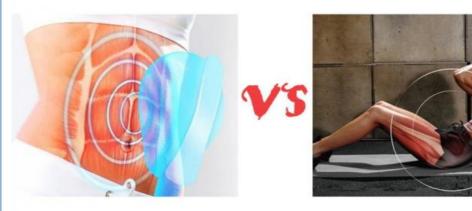



BUILD MUSCLE

The muscle contracts 30000 times withhigh frequency and intensity, so asto train and increase muscledensity and volume



The ultimate contraction of muscle needsa large amount of energy supply, so thefat cells beside the muscle are alsoconsumed, leading to naturalapoptosis and effectivereduction of fatthickness



Treatment

Safe Treatment

Suitable for everyone - No anaesthesia - No surgery

Pulsed Electromagnetic Fields have been used in medicine for many years and are proven and effective technologies

Short Time requirement and Instant results

1 session = 30 min. / treatment area

2 sessions / a week

Total 4 sessions as a course.

And continue to improve for several weeks after treatment.

Before and After



hip bone, or there is more fat distribution on both sides of the waist,so the overall appearance is square, and it is closer to "straight up and straight down" rather than a smooth curve from waist to hip.

mellow visual effect from the back. As long as it's not too much fat, the hip shape can be kept better

storage around the hips and thighs is mainly caused by female hormones, As you age, decreased hormones may cause fat to tend to be stored around the abdomen.

and older women. The main feature is large at the top and small at the bottom.If you don't pay attention to muscle training, this hip type will begin to change to a loose shape.

Function

Function

* BUILD MUSCLE

* REDUCE FAT

The ultimate contraction of muscle needsa large amount of energy supply, so thefat cells beside the muscle are alsoconsumed, leading to natural apoptosis and effective reduction of fatthickness

* BODY SHAPE

Exercising the abdominal muscles, shaping the vest line / exercising the hip muscles, creating the peach hips / exercising the abdominal oblique muscles, and shaping the mermaid line.

Female who seeks improvement in muscle and fat----BUTT LIFT, abs.

Models, Idols, Celebrities ---- contouring a sexy figure



male who seeks improvement in muscle and fat---muscle building, Chocolate abs.





Comparison before and after treatment Before After the fourth treatment Before 8 weeks after the fourth treatment Before 4 weeks after the fourth treatment Before 4 weeks after the fourth treatment



Our Advantages

Professional 16 years manufacturer.

The 15th year gold supplier on alibaba.

Facotry is also approved by third party"Bureau Veritas and TUV "

If you want to get qucik reply, please contact me by whatsapp: +8618754414717

DUST-FREE WORKSHOP



LOREM IPSUM DOLOR SIT AMET, CONSECTETUR ADIPISCING ELIT, SED DO EIUSMOD TEMPOR INCIDIDUNT UT LABORE ET DOLORE MAGNA ALIQUA. QUIS IPSUM SUSPENDISSE ULTRICES GRAVIDA. RISUS COMMODO VIVERRA MAECENAS ACCUMSAN LACUS VEL FACILISIS.



Weifang KM Electronics Co., Ltd.



13606464486



sales@wfkmdz.com



wfkmdz.com

D3-H-0, D3-H-17, D3-H-18, No. 7999, East Health Street, Yongchun Community, Qingchi Street, High-Tech Zone, Weifang, Shandong, China